

# Water purification tablets 100 pcs

**Continuous use** (use after mounting the bottle and basic cleaning has been performed).

1. Dissolve 1 Aquatab tablet in 1 liter of water every time you fill the bottle.
2. Change water every morning in the unit's hose system by flushing each hose / tap for at least 30 sec.

*Tip! Prepare an extra bottle to save time when changing.*



## Special measures for longer breaks (holidays, long weekends etc.)

## Before the break:

1. Dissolve 3 Aquatab tablets in 1/2 liter of water.
2. Rinse the system as in a normal morning routine and leave the unit this way during the break (leave the bottle mounted).

## After the break:

1. Change water with the normal dosage of Aquatabs (1 tabl / liter).
2. Rinse as usual in the morning routine and then continue as usual.

**ACCESIA®**



**The Vet Dentistry Company**