UNIVET

EOS
next

Quick Guide

Thank you for choosing us

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Congratulations on your new EOS Next.

Register your product now to activate the warranty and access content reserved exclusively for the Univet Loupes community. Keep the original packaging; in the case of a return or maintenance, it will be necessary to send the product to your authorised dealer.

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www.univetloupes.com/registration-eos-next

Made in Italy Made by Univet

Italian manufacturing tradition defines the shapes and high-end materials used in our glasses. Univet loupes are not just tools — they're a testimony to the excellence of Italian craftsmanship.

Since 1997, Univet has been an ambassador for the style, quality, and luxury typical of the Italian fashion and design world. Throughout our history, we've created iconic shapes that distinguish our work and have reimaged the idea of professional eyewear.

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www.univetloupes.com/userguide-eos-next

Go to this link or scan the QR Code to get information in other languages.

Light Design

Our LED system molds and directs light into a precision tool. Its shapes and technology combine to create an extension of your magnification system.

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BLUE LIGHT

compliance

All Univet headlights are tested for the photobiological impacts of prolonged exposure to LED-emitted blue light. Our headlights are designed in accordance with the international standard IEC 62471:2006, which specifies exposure limits that prevent adverse health effects.

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
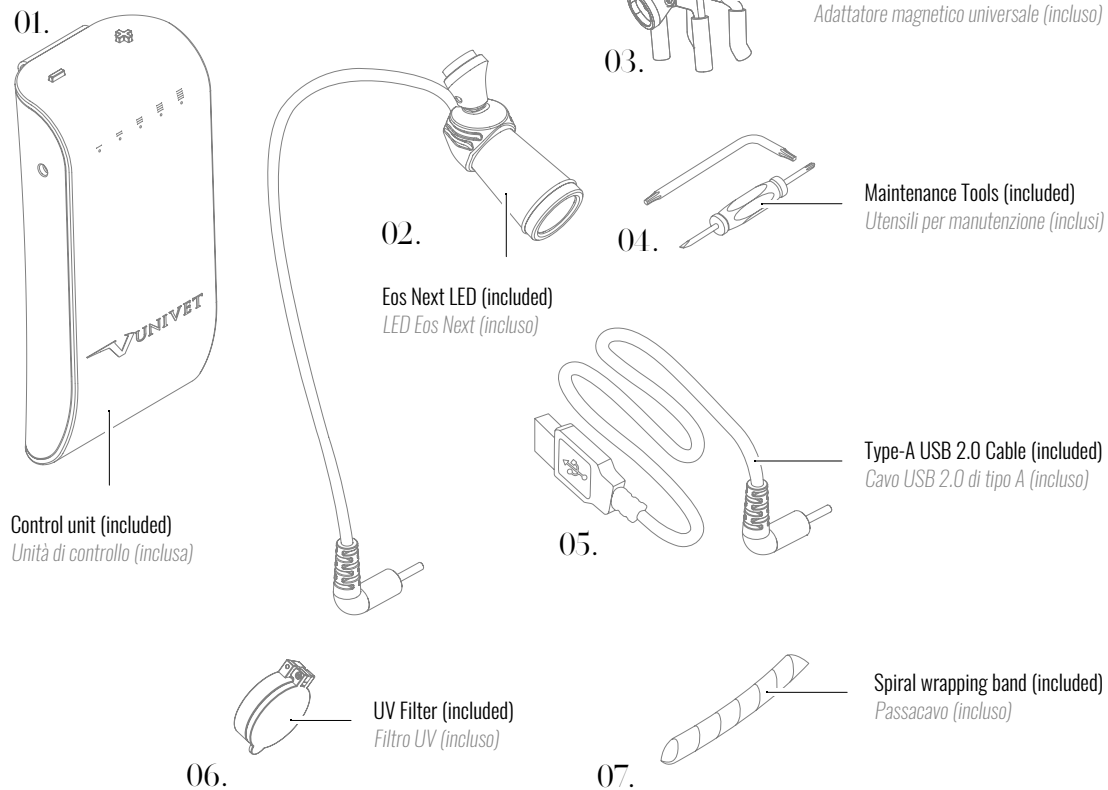
Where to find the components

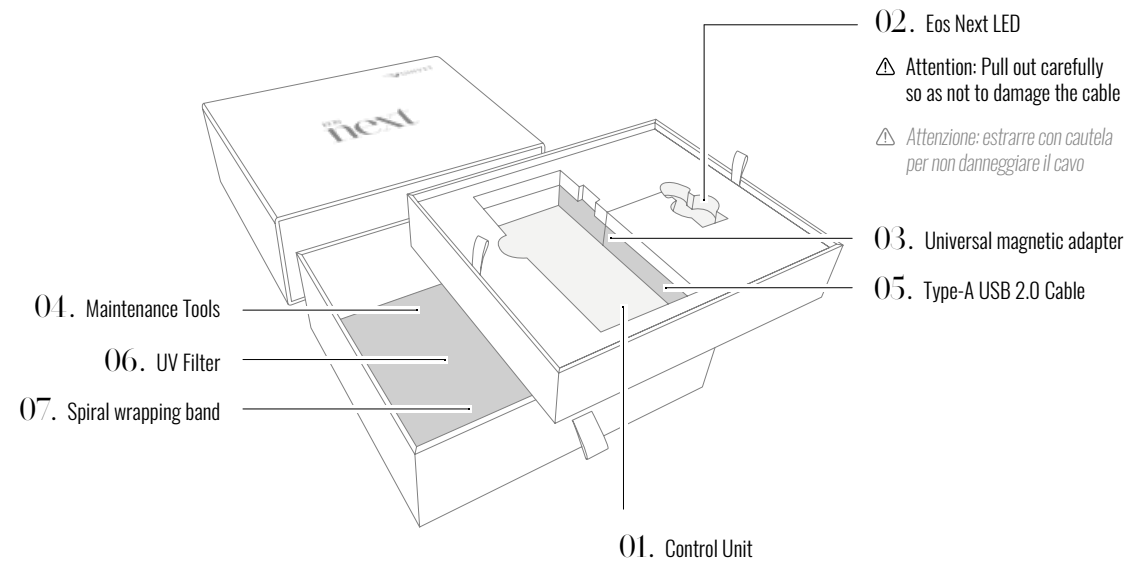CAUTION

Follow all steps in the sequence indicated for the optimum correct result.

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
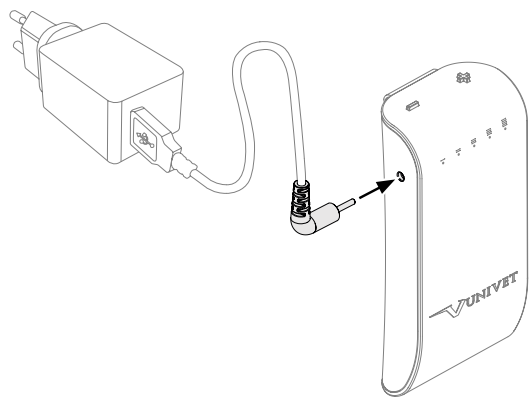
Control unit recharge


Control unit charging time: 6 hours

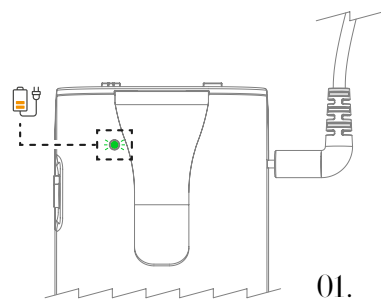To recharge the control unit, you need to insert the jack plug of the USB cable into the appropriate side hole. Then connect the cable to a power supply unit with a Type-A USB port (see specifications on the opposite page). While charging, the charge status indicator on the back of the control unit will flash green (Fig.01) and turn solid green when fully charged (Fig.02).

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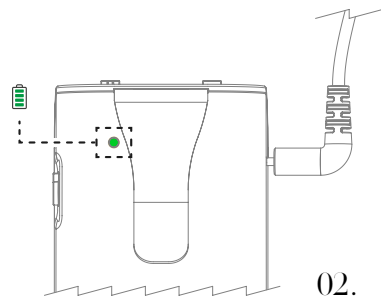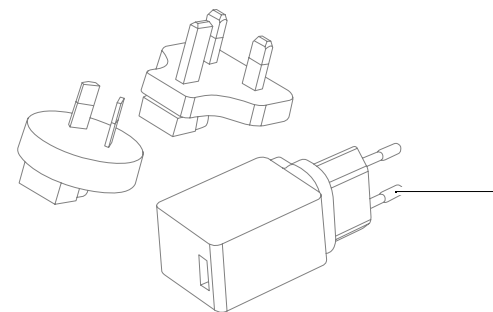
USB power supply unit

You can use a standard power supply with a USB Type-A port and 2A current power supply unit for correct charging. If your power supply unit has a 1A power supply, the device's charging time could last longer than 6 hours. If necessary, it is possible to purchase the Univet Universal Battery Charger (code KZA004318).

CAUTION

When using the EOS Next for the first time, we recommend that you recharge the control unit included in the package.

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Key to the battery charge status

Battery charge status indication

Quickly press the ON/OFF button with the device off to check the battery charge status. The indicators on the front will turn on, indicating the device's charge level (C).

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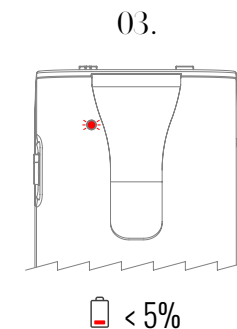
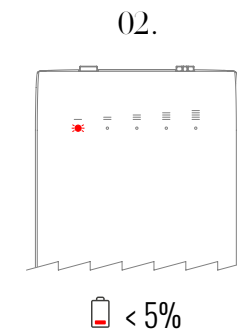
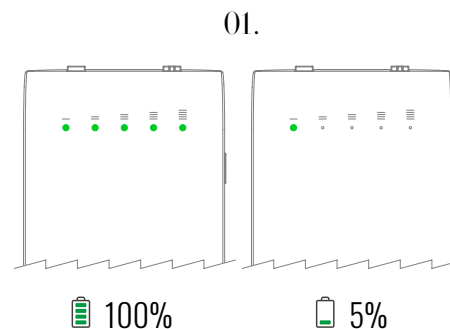
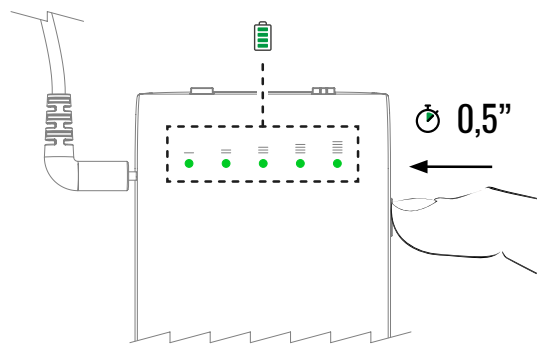
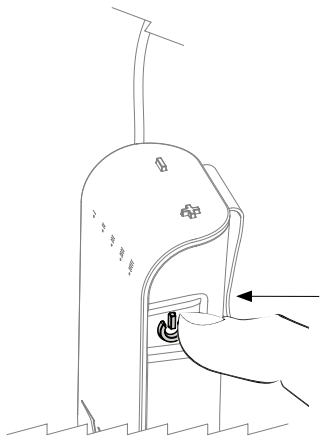
Battery charge levels

The front indicators (Fig. 01) are green if the device is charged, according to 5 charge levels ranging from 100% to 5%. With less than a 5% charge, the first front indicator (Fig.02) and the rear one (Fig.03) will flash red, warning you of the imminent switch-off of the device.

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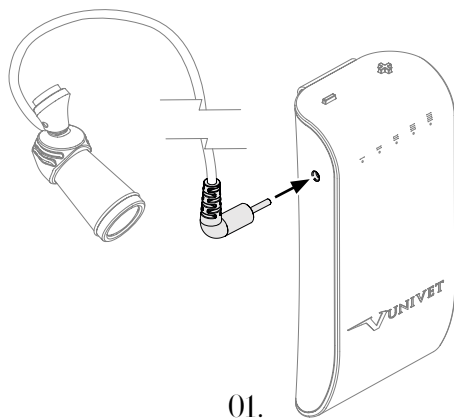
Turning the device ON/OFF

LED headlight connection

Connect the LED headlight to the control unit (fig.01) by inserting the jack plug at the end of the LED cable into the appropriate side port (the same port used for recharging).



Once the LED headlight is connected to the control unit, press and hold the ON/OFF button for 3 seconds to switch on the illuminator (fig.02). To switch it off, press the ON/OFF button again for 3 seconds.

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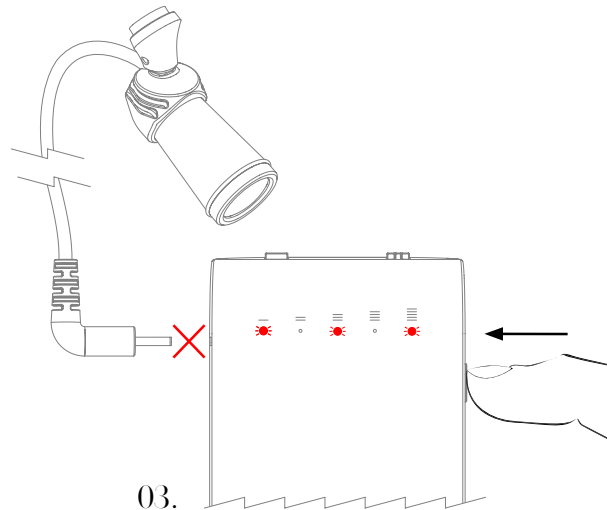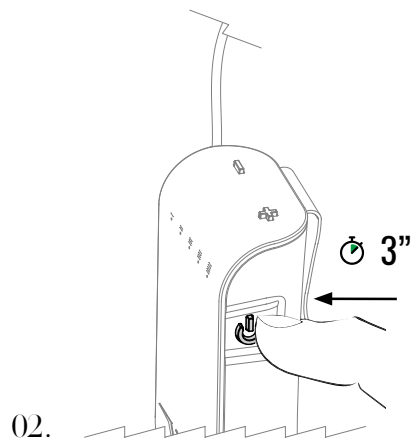ATTENTION: Error signal

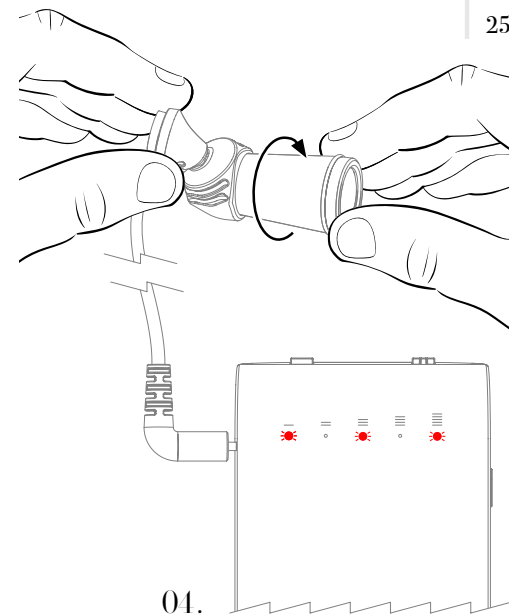
If the illuminator is switched on without connecting the LED headlight to the control unit, the front indicators will light up red, indicating an error.

When you connect the LED to the control unit, but the front indicators light up red, indicating an error, it is recommended to screw the LED back onto the base at the end of the cable (Fig.04).

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LED light intensity

27

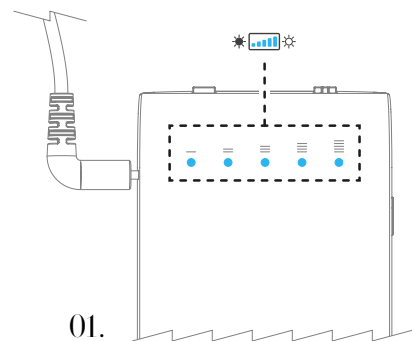
Legend of the light intensity levels

Light intensity level indication

Once the illuminator has been switched on, the indicators positioned on the front (Fig.01) will turn blue, indicating the level of luminous intensity of the LED headlight.

Switching from luminous Intensity to Charge Status

With the device switched on, it is possible to switch from the indication of the luminous intensity to that of the battery charge status by quickly pressing the ON/OFF button (Fig.02)

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
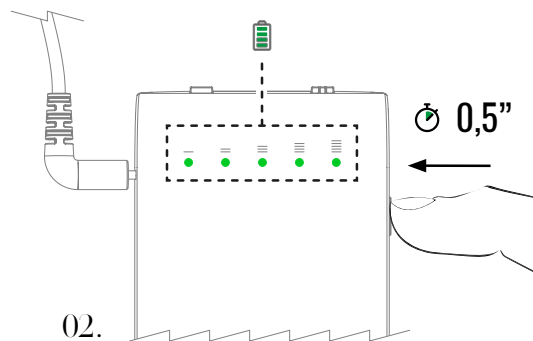
Light intensity levels

The light emitted by the LED headlight is adjustable according to 5 levels of intensity thanks to the minus (-) and plus (+) keys located in the upper part of the control unit. Each time the minus (-) key is clicked, the light intensity level will gradually decrease to a minimum of 1 (A). On the contrary, with each click of the plus (+) key, the brightness level will gradually increase up to a maximum of 5 (B). The brightness level selected before switching off the device will be memorised and re-proposed when turned on again.

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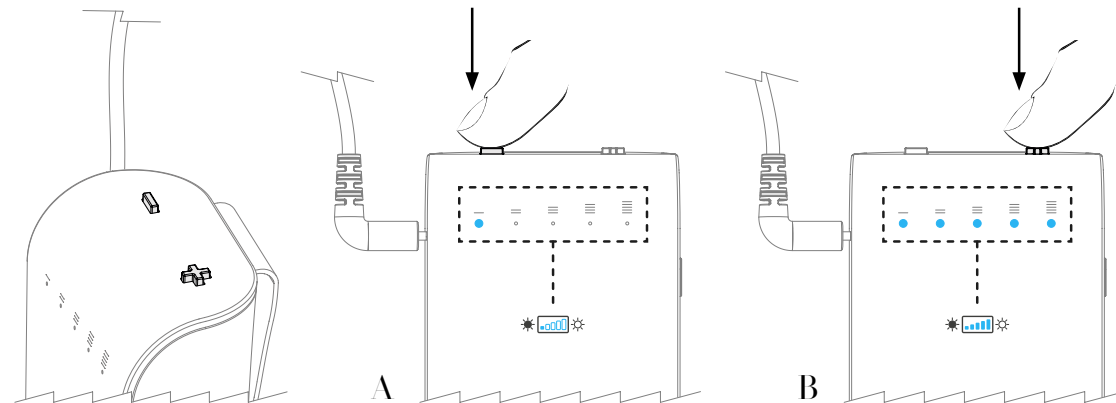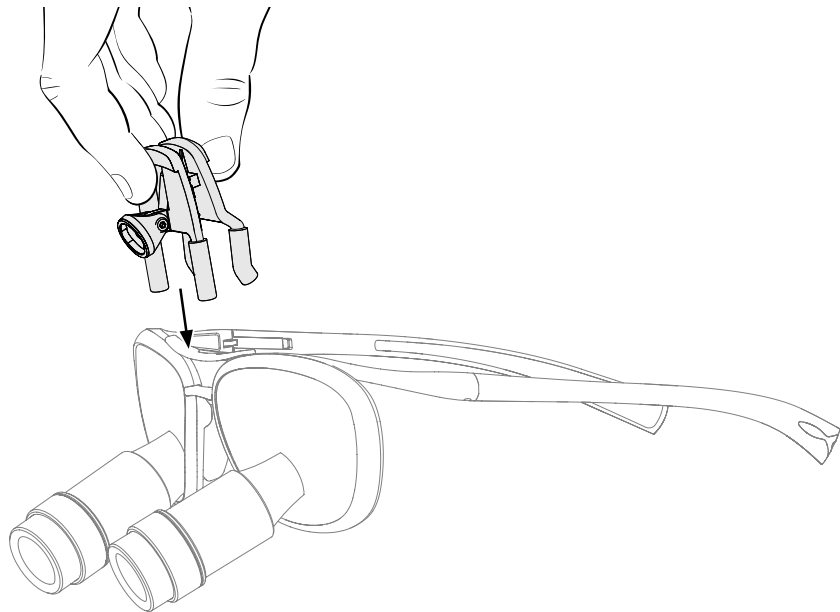
Assembly & Adjustments

31

Attachment to the magnifying system

Hook the universal adapter

Open the clip of the adapter (included in the package) and place it over the nose bridge of the loupes.

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

LED headlight fastening to the universal or custom adapter

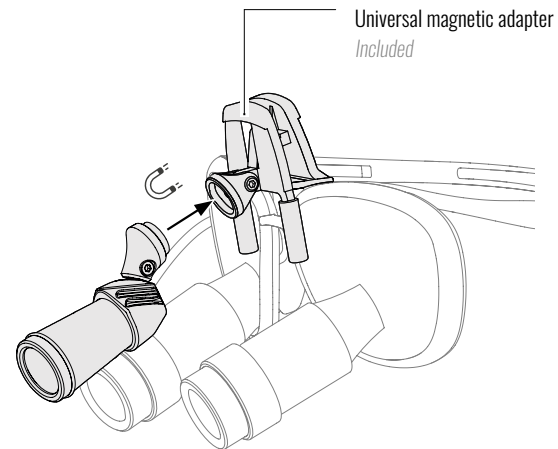After attaching the adapter to the loupes, attach the LED headlight to the adapter using the magnet. The operation can be performed in the same way on both the universal and the custom adapter.

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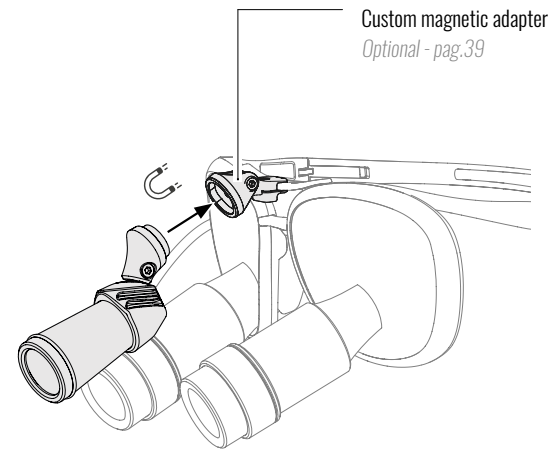
Led adjustment

Adjusting the LED

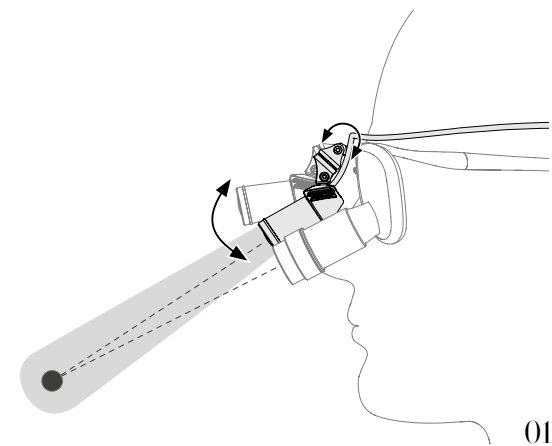
We advise you to adjust the inclination of the LED headlight so that the light spot illuminates the field of view, acting on the joint of the magnetic coupling and the joint of the LED headlight. Once the correct position has been established (Fig.01), we suggest tightening the screws (Fig.02) using the Allen key (included in the package).

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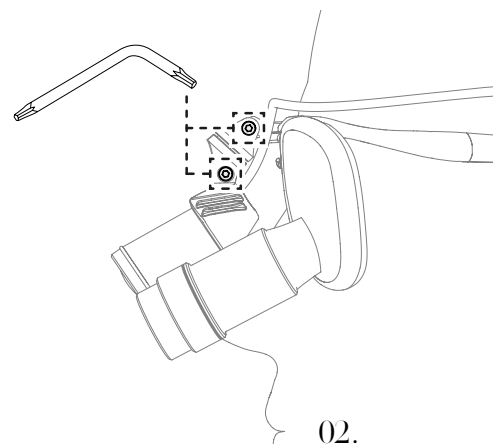
Additional assembly

Use of the cable gland

To avoid hindrances caused by the LED cable during its use, we recommend you fasten it through the cable glands to the frame of your loupes (for more information download the quick guide dedicated to your Loupes at the page www.univetloupes.com/userguide). In the case of using non-Univet loupes, it is possible to use the spiral wrapping band included in the package.

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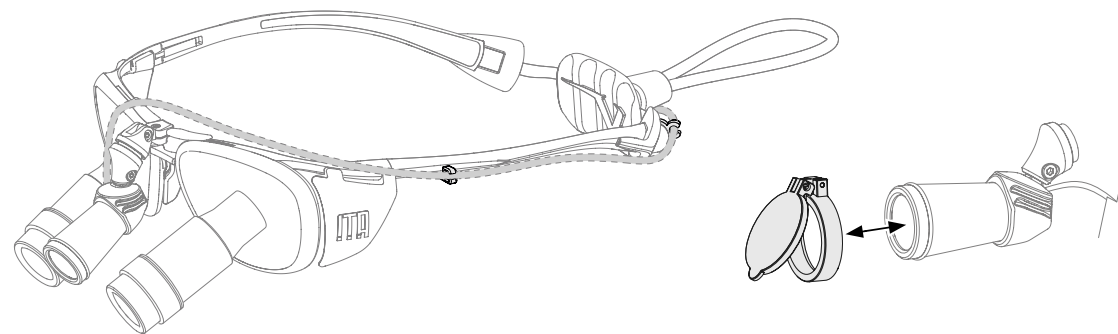UV filter fastening

With a light pressure fasten the UV filter (included in the package) on the front of the LED headlight.

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Optional

Cod. KZA004318
Univet Universal Battery Charge
Caricabatteria Universale Univet

Cod. KZ36818.00
LED EOS Next spare cable
Ricambio cavo LED EOS Next

Cod. KZ41718.0A
ITA Loupes
Custom Headlight Adapter
Attacco illuminatore

Cod. KZA005531
ONE/LOOK&COOL Loupes
Custom Headlight Adapter
Attacco illuminatore

Cod. KZ25818.0A
TECHNE/TECHNE RX Loupes
Custom Headlight Adapter
Attacco illuminatore

Cod. KZ25918.0A
ASH Loupes
Custom Headlight Adapter
Attacco illuminatore

Custom magnetic adapter

Optional

Our custom, model-specific magnetic adapters allow you to quickly and comfortably mount your headlight to your Univet loupes, transforming loupes and light into an integrated system.

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Remote Control

UNIVET CONNECT APP

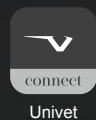
Through the Univet Connect app, you can remotely control your EOS Next by using an iOS or Android smartphone. Scan the QR Code and discover all the functions.

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FIND OUT MORE

univetloupes.com/univet-connect

All specifications and descriptions are subject to change without notice. All rights reserved. Text, images, graphics are subject to copyright and other protective laws. The content of this publication may not be copied, distributed or modified for commercial purposes without written authorization by Univet Loupes Spa.

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