Univet Connect

Download & Install

iOS

Download and Install the Univet Connect App

Search for Univet Connect in the App Store, where you can download the app for controlling your EOS Next LED device for free.

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Minimum system requirements

iOS operating system: requires iOS 10.0 or later

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App Store

ANDROID

Download and Install the Univet Connect App

Search for Univet Connect in Google Play, where you can download the app for controlling the EOS Next LED device for free.

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Minimum system requirements

Android operating system: requires Android 8.0 or later

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Google Play

Before using the App

Set the power switch on the EOS Next LED illuminator device to on

Connect the LED headlight to the control unit (Fig.01) by inserting the jack plug at the end of the LED cable into the appropriate side port (the same port used for recharging). Once the LED headlight is connected to the control unit, press and hold the ON/OFF button for 3 seconds to switch on the headlight (Fig.02).

For further information on EOS Next, please refer to the Quick Guide supplied with the device.

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
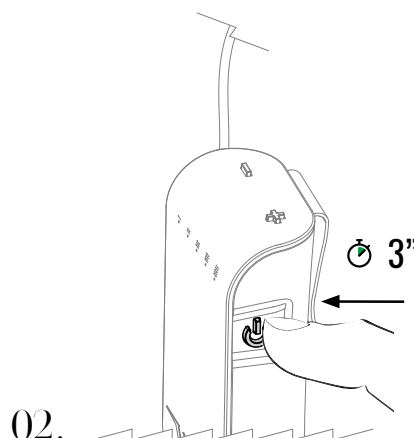
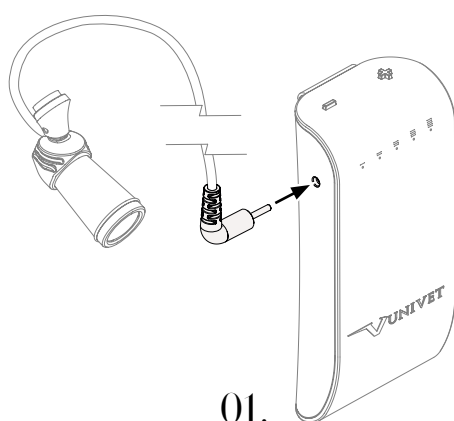
Enable Bluetooth on your Smartphone

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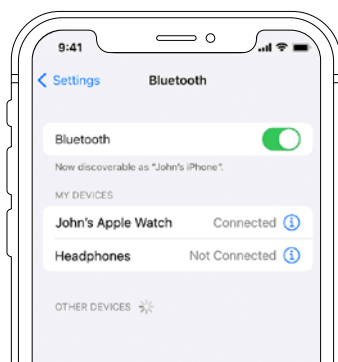Application Settings

Open the Univet Connect app

Click on the Univet Connect app icon to access the "Login" screen. Register a new account by clicking on the "Sign in" (Fig.01). button (Fig.01). This will open the "Create an account" screen (Fig.02), where you enter your data (acceptance of the Univet Loupes Privacy Policy is required). It is advisable to click on the "Remember me" checkbox to save your login information.

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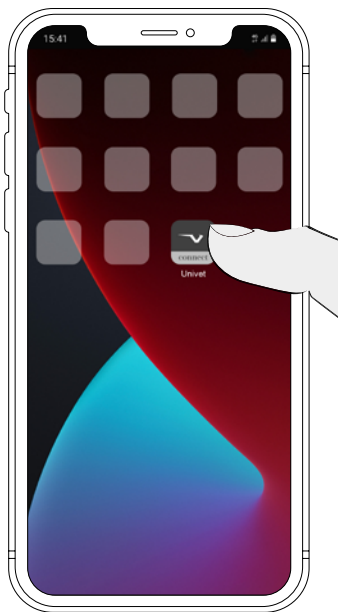
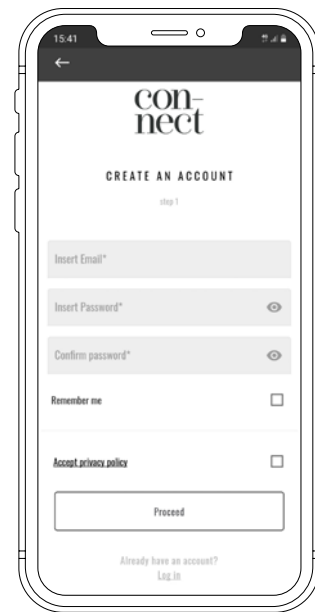
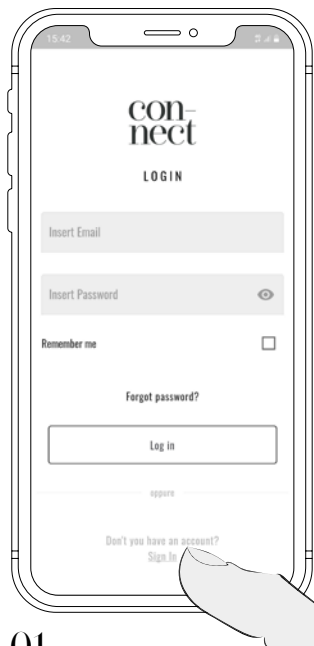
01.

02.

If you already have a Univet account, enter your login information.

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Associate the LED Device

Once Login is completed, you will see a screen with a list of the devices associated with your user profile. If you are signing in for the first time, the list of associated devices will be empty, as shown in the image (Fig.01). Click on the “+” icon at the top-right of the menu and manually enter the Serial Number (Fig.02), which can be found on the back of the Control Unit of the EOS Next LED device.

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04.

At this point, assign a name to the newly associated device. When the procedure has been successfully completed, a “Device added” message will appear on screen (Fig.03). If adding the device is unsuccessful, a “Attention” message will appear (Fig.04). In this case, close and reopen the app, then follow the prompts to associate the device again.

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Control Panel

Select the device to view the Control Panel

Click on the name of the recently added device (Fig.01) to access the "Control Panel" screen where you can: switch the LED device ON/OFF (Fig.02), adjust the 5 light intensity levels (Fig.03), check the battery charge status and remaining battery life (see Key on the following page).

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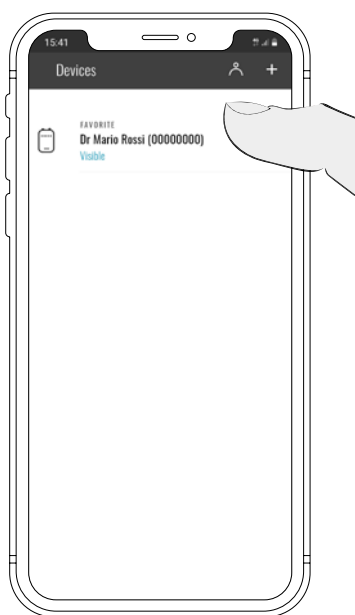
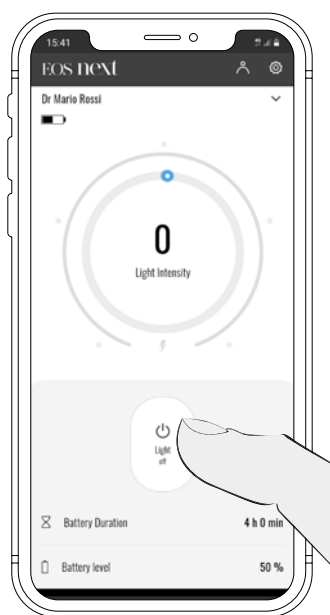
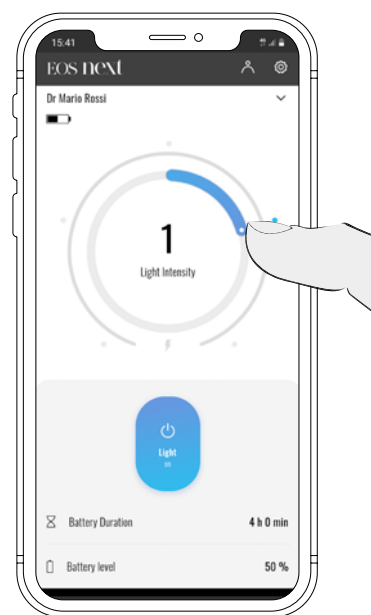
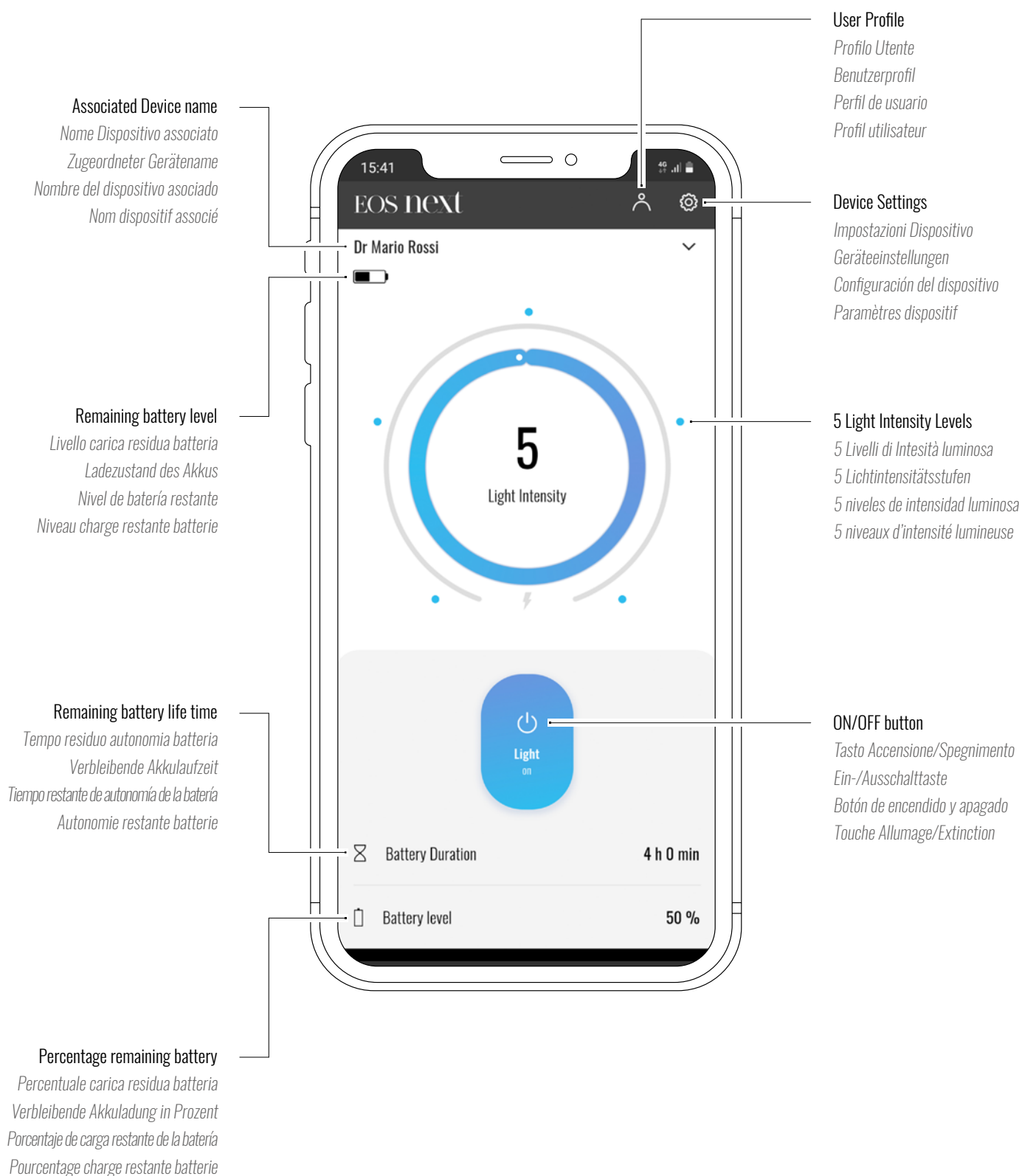
Device Settings

System Information and Updates

Click on the «⚙» icon at the top-right of the menu (Fig.01) to access the properties screen, where you can view all of the information on the associated device and also search for new updates by clicking on «Search updates» (Fig.02).

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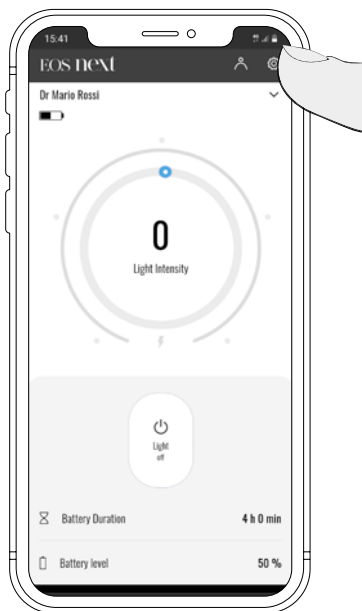
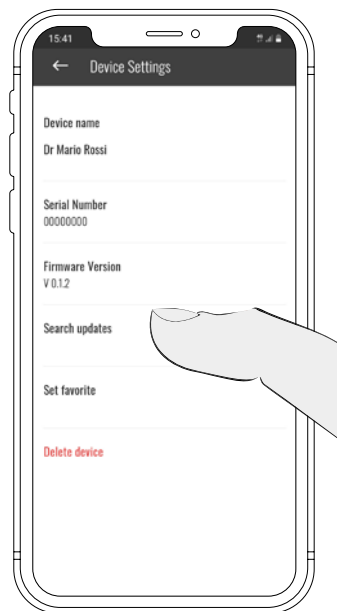
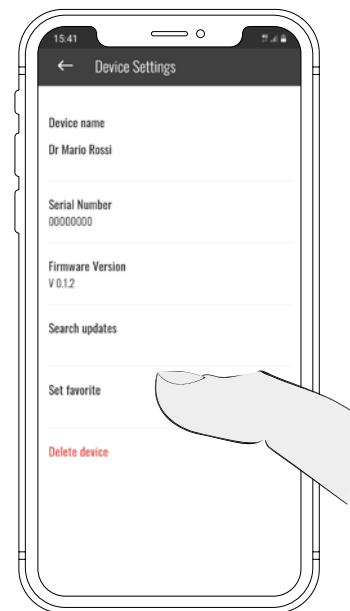
03.

Set Device as Favourite

If more than one EOS Next LED device has been added, click on «Set favourite» to add the selected device to your favourite apps (Fig.03). This will allow you to directly view the «Control Panel» screen when launching the app on your favourite device.

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Delete Device

Removal of an associated LED device from the "Device Settings" screen

If you need to delete a previously associated EOS Next LED device from the "Device Settings" screen (as shown on the previous page), click on "Delete Device" (Fig.01).

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Removal of an associated LED device from the "List of Associated Devices" screen

Alternatively, to delete a previously associated EOS Next LED device, from the "Control Panel" screen, click on the "✓" icon at the top-right of the menu (Fig.02) to access the "List of Associated Devices" screen, then swipe or click to the left of the name of the device you want to remove (Fig.03). At this point, click on the red "X" icon (Fig.04).

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User Profile

Account Information and Change Password

By clicking on the “” icon at the top-right of the menu (Fig.01), you will be able to access the properties screen, where you can view all of your Account information. If you need to change your password, simply click “Modify password” and follow the system prompts on the screen.

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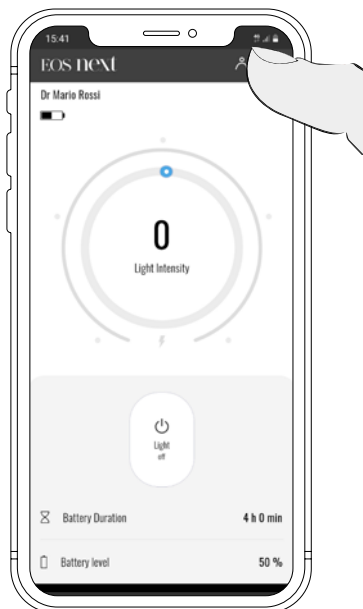
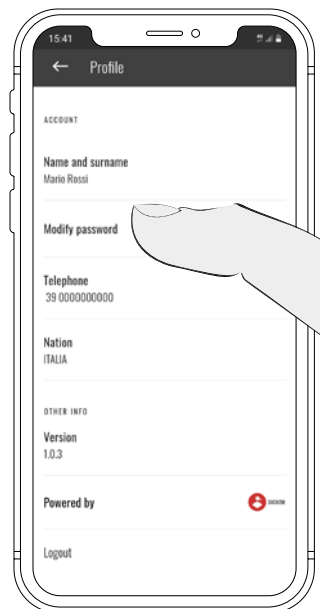
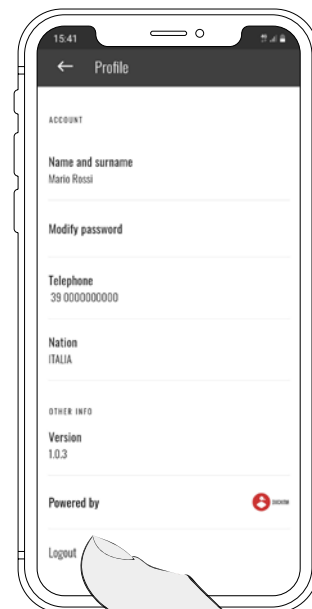
03.

Log out

To disconnect from the app, simply click “Log out” (Fig.03). At this point, the app will return to the “Login” screen where you can set up a new Account.

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