

Equine Dental Speculum

Light Installation instructions:

1. Look for a suitable place on the **left** side of the speculum (horse) to mount the Magnet Holder.
2. Degrease the metal using the alcohol.
3. Dry the metal using paper towel.
4. Make sure the metal is warmer than 20° C
If this is not the case, heat the metal and sticker using a hair blower.
5. Remove the sticker on the back of the Magnet Holder.
6. Stick the Magnet Holder into place and push firmly.
7. If you have a speculum with only round edges and no flat parts, it is advised to use extra tie wrap and/or tape.
8. "Click" the Battery Pack on the Holder by turning it into place. **Attention for the fingers, these are strong magnets, always approach from the side.**
9. Open the speculum and put the Lamp into place.
10. Route the cable so that it cannot be crushed and plug in the connector
The Light is now ready for use.



Note: this device is for professional veterinarian use only. Irresponsible use may result in damage to the device or in worse case injury to the user. In any case of doubt contact the manufacturer. The manufacturer cannot be held liable in case of misuse.

Equine Dental Speculum Light

User instructions:

1. Follow Installation instructions, see installation manual before first use.
2. "Click" the Battery Pack on the Holder by turning it into place.
Attention for the fingers, these are strong magnets, always approach from the side.
3. Open the speculum and put the Lamp into place.
4. Route the cable so that it cannot be crushed and plug in the connector.
5. Switch on Lamp by pushing the metal button once. **Never look directly into the bright light.**
6. An orange light will appear around the pushbutton.
7. This light will blink when the battery is almost empty.
8. Especially when working with power tools, pay extra **attention not to damage the cable.**
9. Before removing the Lamp, always switch off the light by pushing the button again. **The light may only burn when in speculum.**
10. Remove the Lamp by sliding it out of the speculum and stick it under the Battery Pack.
11. Unplug by holding the connector. **Never pull on cables.**
12. Clean the lens after EVERY use, with the same mild disinfectant you use for the speculum between horses. **Keep the lens clean and the connectors dry, ALWAYS.**
13. Remove the Battery Pack from the Holder, by twist/pulling it off.
14. For charging make sure the power is off (no orange light) and plug in the Charger. Full charge takes about 5 hours.
15. For storage in the box: **make sure the battery is not (almost) empty and the power is off.** Take special care that the cables cannot be crushed by closing the lit.

Note: this device is for professional veterinarian use only. Irresponsible use may result in damage to the device or in worse case injury to the user. In any case of doubt contact the manufacturer. The manufacturer cannot be held liable in case of misuse.