






FKG ROOTER X3000


Correct settings for the Vet Flex&Preserve system

1. Press the purple powerswitch to start the machine: 
2. Use + and - buttons until you get to program **M1**.




Check/modify User Program parameters:


3. To select desired parameter press button **P**: 
4. To modify the parameter setting, use buttons + and -.
Press the purple powerswitch:  or wait 5 seconds to confirm the change.
5. These are the correct parameters you should use for the Vet Flex&Preserve file system:

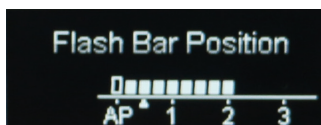


Flash bar position

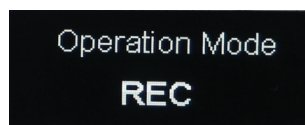


Operation mode: REC

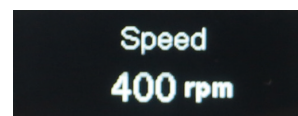

Speed: 400 rpm

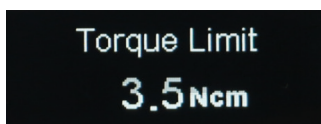


Torque Limit: 3.5 Ncm

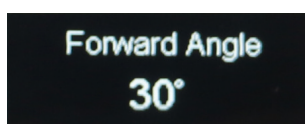


Forward Angle: 30°




Reverse Angle: 120°

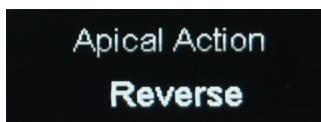

Apical Action: Reverse

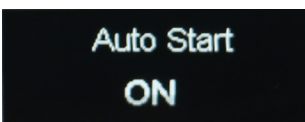

Auto Start: ON

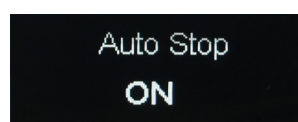

Auto Stop: ON