

# Unit Clean

## MANUAL



Cleaning system for your dental unit  
easy - scientific - clean water

# UNIT CLEAN - CLEANING ONCE A MONTH

If you have a water bottle.

## Unit Clean 1

### Day 1 after the last patient

1. Unscrew the water bottle from the unit and empty it.
2. Empty Unit Clean 1 into the water bottle.
3. Screw on the bottle.
4. Press the 3-way syringe, turbine etc. so that the colored liquid comes out of the instruments. Leave the liquid in the instruments overnight.



### Day 2 before the start of the working day

1. If needed, rinse away remaining liquid.
2. Fill the bottle with clean water.
3. Press the 3-way syringe, turbine etc. so that all colored liquid leaves the unit.
4. Start working with your unit.

## Unit Clean 2

### Day 2 after the last patient

1. Unscrew the water bottle from the unit and empty it.
2. Empty Unit Clean 2 into the water bottle.
3. Screw on the bottle
4. Press the 3-way syringe, turbine etc. so that the colored liquid comes out of the instruments. Leave the liquid in the instruments overnight.



### Day 3 before the start of the working day

1. If needed, rinse away remaining liquid.
2. Fill the bottle with clean water.
3. Press the 3-way syringe, turbine etc. so that all colored liquid leaves the unit.
4. Start working with your unit.

You can now work with a clean unit for a month.

# UNIT CLEAN - CLEANING A STRONGLY CONTAMINATED UNIT

If you have a water bottle.

## Unit Clean P + buffert

### Day 1 after the last patient

1. Unscrew the water bottle from the unit and empty it.
2. Mix Unit Clean P with the Buffert (small tube).
3. Pour the mixed liquid into the water bottle.
4. Screw on the bottle. Press the 3-way syringe, turbine etc. so that the colored liquid comes out of the instruments.
5. Leave the liquid in the instruments over night.

**NOTE! Max 24 hours.**



### Day 2 before the start of the working day

1. If needed, rinse away remaining liquid.
2. Fill the bottle with clean water.
3. Press the 3-way syringe, turbine etc. so that all colored liquid leaves the unit.
4. Start working with your unit.

Thereafter, cleaning with Unit Clean 1 and Unit Clean 2 should be done once a month.

### Contact us:

Accesia AB

Tel: +46 (0) 35 397 97

E-mail: [info@accesia.se](mailto:info@accesia.se)

[www.accesia.se](http://www.accesia.se)